voco**Grandio®SO****GB**

In accordance with DIN EN ISO 4049

Instructions for use**CE 0482****Product description:**

GrandioSO is a light-curing, radiopaque restorative. **GrandioSO** contains 89% w / w inorganic fillers in a methacrylate matrix. **GrandioSO** is used with a dentine-enamel bond and is available in syringes and caps for direct application.

Shades:

A1, A2, A3, A3.5, A4, B1, B2, B3, C2, D3, Incisal, OA1, OA2, OA3.5, BL, GA3.25, GA5

Indications:

- Class I to V restorations
- Reconstruction of traumatically damaged anterior teeth
- Facetting of discoloured anterior teeth
- Correction of shape and shade for improved aesthetic appearance
- Locking, splinting of loose teeth
- Repairing veneers
- Restoration of deciduous teeth
- Core build-up under crowns
- Composite inlays

Preparation:

Clean the teeth to be treated.

Shade selection:

To select a shade, compare the enclosed shade guide with the cleaned tooth before anaesthesia. Moisten the **GrandioSO** shade sample and tooth and select the shade, preferably in daylight.

Creating a dry working field:

Ensure that the work area is sufficiently dry. Use of a rubber dam is recommended.

Cavity preparation:

In principle, cavity preparation should be carried out according to the rules of adhesive filling therapy and should be minimally invasive to conserve healthy tooth substance. Bevel the enamel margins on anterior teeth and round off the preparation margins on posteriors. Clean and dry the cavity afterwards. Non-carious cervical lesions do not have to be prepared; thorough cleaning is sufficient here.

Lining: A suitable pulp-protective liner should be applied in close proximity to the pulp.

Matrix placement:

Use of a shaping aid is recommended for cavities with an approximal portion. Translucent matrices that are wedged in the approximal area are advantageous. Minimal separation facilitates the shaping of the approximal contact and insertion of the matrix.

Handling:

Allow the material to reach room temperature before use.

Bond material:

Grandio SO is used in the adhesive technique with a dentine-enamel bond. All light-curing bond materials can be used. Follow the respective instructions for use with regard to preparation (etching technique) and application.

Application of GrandioSO:

Apply the selected shade of **GrandioSO** (see shade selection) in layers that are a maximum of 2 mm thick, adapt with a suitable instrument and light-cure afterwards.

Light-curing:

Conventional polymerisation devices are suitable for light-curing the material. The curing time is as follows when using an LED-/halogen light with a minimum light-output of 500 mW / cm²

20 s for shades A1, A2, A3, A3.5, A4, B1, B2, B3, C2, D3, Incisal, BL, GA3.25
40 s for shades OA1, OA2, OA3.5, GA5

The curing time is as follows when using an LED- / halogen light with a minimum light-output of 800 mW / cm²

10 s for shades A1, A2, A3, B1, Incisal, BL
20 s for shades A3.5, A4, B2, B3, C2, D3, GA3.25, GA5
40 s for shades OA1, OA2, OA3.5

Hold the light emission tip of the device as close as possible to the surface of the filling. Otherwise, the curing depth may be reduced. Incomplete curing may lead to discolouration and discomfort.

Finishing:

The restoration can be finished and polished immediately after removing the matrices (e.g., fine or extra-fine diamond grinder, polisher). The tooth should be fluoridated as a final step.

Grandio SO can be used for direct and indirect inlays according to customary methods. The physical stability values can be improved by conventional external (additional) curing.

Information, precautionary measures:

- No known side effects. Sensitisation in hypersensitive patients, however, cannot be excluded.
- **GrandioSO** contains methacrylates and BHT. **GrandioSO** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
- Phenolic substances, especially preparations containing eugenol or thymol, interfere with the curing of filling composites. Avoid the use of zinc oxide eugenol cements or other eugenol-containing materials in combination with filling composites.
- Each **GrandioSO** Caps unit should be used for one patient only and should be applied exerting uniform but not excessive pressure.
- Fillings exposed to occlusal forces should be checked at least once a year for early detection of any changes.

Storage:

Store at temperatures between 4 °C - 23 °C. If refrigerated, the material must be allowed to reach room temperature before use. To avoid exposure to light and potentially resulting polymerisation, syringes should be closed immediately after dispensing. Do not use **GrandioSO** after the expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VOCO GmbH