


■ Plus Wipes Unisepta®

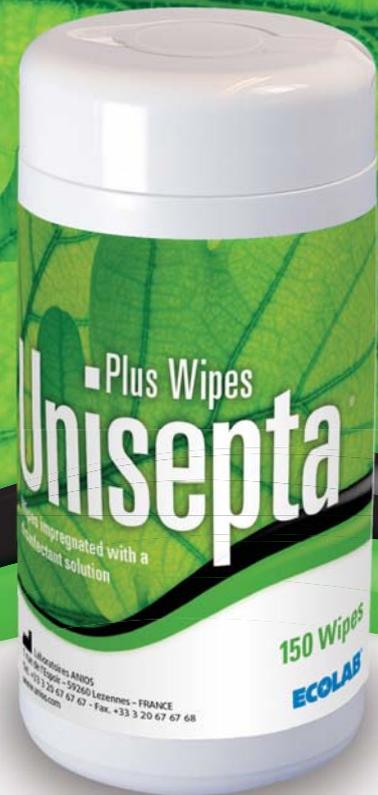

Ready-to-use wipes impregnated with a fast acting disinfecting solution for treating previously cleaned, alcoholresistant, non-immersible and non-invasive medical devices (examples: dental unit, dental curing lamp body, etc.).

Key product features

- Ethanol based
- Contact time: 30 seconds
- Fresh citrus fragrance
- Leave no residue or smearing
- Ideal for use between patients
- Wipe size 130 x 190mm

Plus Wipes Unisepta®

Protocol for use

Before use

Ensure that appropriate gloves are worn.


Check before application that the area to be treated is compatible with alcohol.



Remove the wipe from the container by pulling horizontally and separate with a sharp pull.



Apply the disinfectant wipe to the previously cleaned area, using as many wipes as necessary.

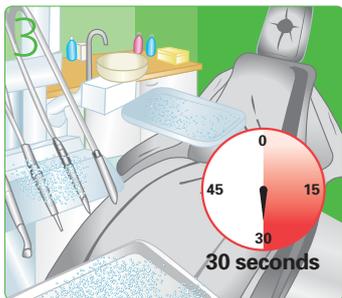

Allow the solution to act for a minimum of 30 seconds. Rinse thoroughly with tap water to remove any trace of the product.




Ensure that the container is closed securely after each use.

Precautions

Dangerous: respect the precautions for use (established according to the European regulations in force for classification and labelling of chemicals). Before each use, read the label and the information concerning the product. Storage between +5°C and +25°C.

Reserved for professional use. Medical device CE 0459.

Efficacy**

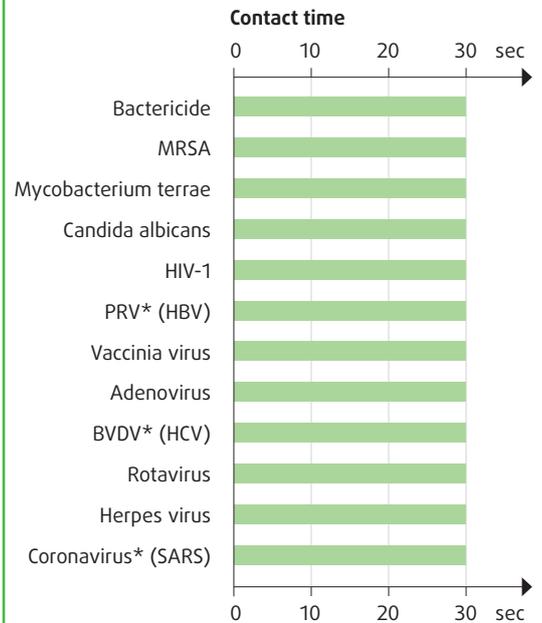

* PRV: Surrogate of Hepatitis B

* BVDV: Surrogate of Hepatitis C

* Coronavirus: Surrogate of SARS

** Efficacy of impregnation solution

Conforms to European standards

Bactericidal and yeasticidal wipes according to EN 16615.

The impregnation solution is:

Bactericide: EN 13727

MRSA: EN 13727

Yeasticide: EN 13624

Tuberculocide: EN 14348

Active on viruses according to EN 14476

Product composition

100 g of the impregnation solution contains: Ethanol (55 g), didecylmethyl oxyethylammonium propionate (0.11 g), synthetic perfume, excipients.

Physical data

Colour: White wipe

Perfume: Fresh citrus fragrance

Shelf Life: 3 years (unopened)

Use within 3 months of opening

Containers: Made of recyclable HDPE material

Packaging

	150 Wipes
Container	✓
Refills	✓