Caution: U.S. Federal laws restrict this device to sale to or on the order of a dentist.

Indications for use: XCP-DS FIT holders are indicated for holding digital sensors for intraoral dental radiographs. XCP and XCP-ORA arms and rings are indicated for positioning and aligning intraoral dental radiographs.

Contraindications:

None known.

Warnings: Sterilize before use. DO NOT use phenol-based glutaraldehyde or ultrasonic cleaners. DO NOT chemiclave, dry-heat sterilize or cold sterilize. Only sterilize with steam autoclaving according to cleaning and sterilization instruction.

Precautions: Follow sterilization instructions carefully to prevent damage to plastic pieces during sterilization. Use sensor holders with a sensor cover. Do not pull on sensor cord during use.

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Step-by-Step Instructions:

See illustrations on reverse side.

1. Place a protective cover over the sensor. ❶
2. Load sensor into appropriate biteblock for the area to be imaged by holding sensor firmly and pressing edge of sensor against the movable clip. ❷
3. Insert opposite sensor edge and release clip. ❸
4. Insert positioning arm into the appropriate color-coded channel of the aiming ring and attach biteblock according to Positioning Guide.
5. Correctly assembled, the sensor should be centered when viewed through the aiming ring.

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