EN

Instructions for use

Whitening gel for whitening of discolored teeth, suitable for minimally invasive treatments using customized bleaching trays. Brilliant Smile certified staff can recommend appropriate duration of the treatment. Recommended treatment duration can be found on the syringe.

Fields of Application

- Teeth whitening vital, discolored teeth
- Teeth whitening non-vital teeth
- Whitening of teeth discolored by medication

Contraindications

- Known allergy or intolerance to any of the ingredients
- Pregnancy and lactation

Use

1. Clean the teeth with Dental Cleanser before treatment.
2. Break off the cap on the syringe. Add a small amount of gel in the hollow portion of the bleaching tray or apply evenly directly on the teeth's surface. One syringe is sufficient to fill the two bleaching trays at least two times. Partially filled syringes can be resealed with the broken cap.
3. Do not fill the trays with excessive amounts of whitening gel, as this does not improve or accelerate the whitening process.
4. Insert the bleaching trays in the mouth.
5. Remove any excess gel from your gums or other soft tissue in the mouth.
6. Treatment duration is determined by Brilliant Smile certified staff.
7. Do not eat, drink or smoke while you whiten your teeth.
8. Remove the trays after treatment and clean them thoroughly with warm water and a toothbrush. Dry them thoroughly and keep them in the tray case. Rinse your mouth thoroughly after treatment to remove any gel residue.
9. Continue the treatment as the Brilliant Smile certified staff's prescribes.

After treatment

During treatment and 72 hours after, avoid acidic foods and drinks (cola, citrus drinks and fruit), as these can cause hypersensitivity. Strong colored drinks (coffee, tea, red wine) and food (chocolate, tomatoes, curry, oranges, etc.) and tobacco should also be avoided during the same period of time to avoid

discoloring of the teeth.

Noticeably whiter teeth can usually be seen after approx. 1 week. For persistent stains, the Brilliant Smile certified staff may extend the treatment period further. Depending on the tooth structure and personal habits (consumption of discoloring food, smoking), the result can last for several years. By using Brilliant Smile Whitening toothpaste and mouthrinse, you may extend duration of the results even longer. Treatment may be repeated at a later time without causing any damage to the tooth enamel. You may want to save the bleaching trays after treatment for later use.

Information and Precautions

Dental fillings and crowns, bridges and veneers made of composite or metal are not affected. These can be replaced if needed to match the new whiter teeth. It is advisable to wait for a period of two weeks before such a replacement or restoration.

Potential side effects

In some cases the teeth may become slightly sensitive to hot and cold during treatment and a couple of days after treatment is completed, manifesting in shooting pains and hypersensitivity. Temporary gingival irritation during treatment may also occur if too much gel is used.

These side effects may be minimized, by reducing the amount of bleaching gel or by discontinuing treatment. Please consult Brilliant Smile certified staff if problems persist or if bleaching trays do not fit. White spots on the teeth's surface may temporarily become more visible during treatment and a couple of weeks after. These are caused by transient demineralization and usually disappear again.

Warning

The gel contains hydrogen peroxide, urea peroxide, glycerine, acrylic polymer, potassium nitrate, sodium fluoride and flavorings. Avoid contact with eyes, skin and mucosal tissue. In case of contact, rinse with water. If contact with the eyes occurs, you should consult a doctor as well. Swallowing large amounts of gel may cause heartburn or nausea in susceptible patients.

Storage

Gel syringes should be stored in the refrigerator. Do not freeze syringes. Protect from heat and direct sunlight. Do not use after the expiration date. Keep away from children!

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