3M ESPE

Ketac™ Molar Aplicap™

Glass Ionomer Restorative Material

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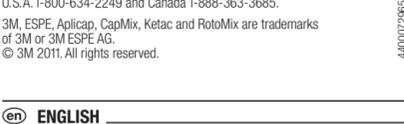
ENGLISH

Product Description

Ketac™ Molar Aplicap™, manufactured by 3M ESPE, is a glass ionomer cement in a capsule. On account of its chemical adhesion to the dental enamel and dentin, a gentle preparation and an especially tight seal at the margins is possible. The product is available in the following shades: A1, A3, A4, and dark yellow opaque (DYO). The material can be applied without lining; it releases fluoride ions, and is radiopaque. The dispensable quantity of a capsule is at least 0.1 ml. For details on Ketac™ Conditioner, CapMix™, RotoMix™, Aplicap™ Activator and Applier and Ketac™ Glaze, all products manufactured by or for 3M ESPE, please refer to the respective information for use.

☞ The information for use of a product has to be kept for the duration of the application.

Indications

- Linings for single-surface and multiple-surface composite fillings
- Core build-up prior to crown placement
- Primary tooth fillings
- Single-surface fillings in non-occlusal load-bearing areas
- Cervical fillings, if aesthetics is not the prime consideration
- Uniplanar and multi-planar temporary fillings

Precautionary measures

3M ESPE MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Preparation

- Remove carious tooth substance only; undercuts are not required.
- Do not prepare thin beveling margins. A wall thickness of 0.5 mm has to be secured for the material.

Pulp Protection

- In order to prevent pulpal irritation cover areas in close proximity to the pulp with local application of a calcium hydroxide material.

Conditioning

- For optimal chemical bonding to the teeth the smear layer produced in the preparatory steps must be carefully removed by applying Ketac Conditioner, manufactured by 3M ESPE, to the prepared surfaces and allowing the substance to react for 10 sec.

Then rinse with copious amounts of water.

- Blow the cavity dry in only 2–3 short intervals with water and oil-free air or dab dry with cotton pellets. Do not over-dry! **The cavity should be just dry that the surface has a matt shiny appearance.** Excessive drying can result in post-operative sensitivity after filling.
- Avoid further contamination.

Activation

- Place the Aplicap™ Activator on a sturdy surface and insert the Aplicap capsule in the activator.

- Using the ball of your hands, depress the activator lever firmly and completely to the stop and hold it down for 2 to 4 seconds.

- Firmly and completely depressing the lever to the stop and holding it in this position is the only way to ensure that all of the liquid is pressed into the powder. If inadequate pressure is applied or if the lever is not depressed completely, it is possible that too little liquid will be mixed with the powder, increasing the viscosity and changing the product characteristics.

Mixing

- Mix the capsules at approx. 4,300 rpm, in a high frequency mixing device such as CapMix™ (for 10 sec) or the RotoMix™ rotating mixer (for 8 sec).

Application

- Apply with the Aplicap™ Applier. Both the Aplicap Activator and the Aplicap Applier are color-coded in red.
- Ensure that there is no contamination from water, saliva, etc., entering the working area for the entire duration of application; the working area must be kept dry.

Attention

- By all means prevent waiting times between activation, mixing, and application because the beginning of setting of the material will render difficult or prevent the application.
- Ketac Molar Aplicap clings to metal instruments, and for this reason should be washed off with cold water before setting.

Times

The following times apply for an ambient temperature of 23° C/73° F:

	min:sec
Activation	0:02
Mixing in RotoMix	0:10
Mixing in high-frequency mixer e.g. CapMix	0:15
Working after start of mixing	2:00
Setting after start of mixing	4:30

Setting is accelerated over 23° C/73° F. An overextended working time will cause the loss of adhesion to the dental enamel and the dentin.

Shorter mixing times will extend, longer mixing times will shorten the working and setting times. The mixing times can be chosen between 8 and 15 sec in high-frequency mixers and between 7 and 12 sec in the RotoMix.

Filling protection

Glass ionomers go through two phases of setting, and are susceptible to water during the first setting period. Subsequently, dehydration may be possible, which can be avoided by applying Ketac Glaze directly after contouring.

- Apply Ketac Glaze to all free surfaces of the glass ionomer cement, and light-cure for 10 sec.

The inhibition layer remaining after the setting of Ketac Glaze may obstruct the setting of the additional type silicone impression materials. For this reason remove this layer or apply a polyether impression material.

Finishing

- Further finishing with Arkansas stones, fine-grain diamonds, abrasive discs with successively decreasing grain size or silicone polishers should be started 4.5 minutes after start of mixing at the earliest.
- If desired, apply Ketac Glaze again for surface sealing and light-cure for 10 sec.

Storage and shelf-life

Store the capsules in the blister packs.

Use within 1 month maximum once the blister pack has been opened!

Store the product at 15–25° C/59–77° F.

Do not use after the expiry date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manu- facture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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