Content

English	2	Bahasa Malay	110
Deutsch	8	Malti	116
Français	14	Nederlands	122
Italiano	20	Norsk	128
Español	26	Polski	134
Български	32	Português	140
Česky	38	Română	146
中文 (中华人民共和国)	44	Русский	152
Dansk	50	Slovenčina	160
Eesti	56	Slovenščina	166
ελληνική	62	Srpski	172
תּוּרְכִי	68	Suomi	178
Hrvatski	74	Svenska	184
Íslenska	80	ภาษาไทย	190
한국어	86	Türkçe	198
Latviešu	92	Українською мовою	204
Lietuvių	98	Distributors	210
Magyar	104	Space for notes	217

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Collagen Matrix

Composition

Geistlich Mucograft® is a collagen matrix obtained by standardized, controlled manufacturing processes. The collagen is extracted from veterinary certified pigs and is carefully purified to avoid antigenic reactions. Geistlich Mucograft® is sterilized in double blisters by γ -irradiation. STERILE R

Geistlich Mucograft® has a bilayer structure. The compressed layer consists of compact collagen which has cell occlusive properties and allows tissue adherence as a prerequisite for favorable wound healing. This layer has a smooth texture with appropriate elastic properties to accommodate suturing. A second layer consists of a thick, porous collagen spongy structure. This spongy surface

is placed next to the host tissue to facilitate organization of the blood clot and to promote formation of new blood vessels besides to promote tissue integration.

The matrix is made of porcine collagen without further cross-linking or chemical treatment.

Properties / Action

The low antigenicity and excellent biocompatibility favour the use of Geistlich Mucograft® in dental and maxillofacial surgery. The long-fibred microstructure of Geistlich Mucograft® readily absorbs fluid. The matrix retains its structural integrity even when wet. Due to the natural origin of the collagen a slight bulging and variations in the thickness

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of Geistlich Mucograft® matrix may be noted. Fixation by sutures or pins is possible. The coherent collagen fibres swell and form a unified basic tissue structure. As a result, the matrix adheres well to the surrounding and underlying tissues.

Indications

Geistlich Mucograft® is intended to be used for various soft tissue augmentation indications in the oral cavity, e.g.

- > Guided tissue regeneration procedures in periodontal and recession defects
- > Reconstruction of the alveolar ridge
- > Augmentation around implants and natural teeth
- > Covering of extraction sockets
- > Covering of bone defects after root resection, cystectomy, removal of retained teeth
- > Cleft closure surgeries
- > GBR (Guided Bone Regeneration) procedures

Instructions for Use

The general principles of sterile handling and patient medication must be followed when using Geistlich Mucograft®:

- > The defect is exposed by means of (a) properly prepared flap(s) and the usual surgical procedures in creating a properly prepared site are undertaken.
- > The defect is filled, if needed, with a space-maintaining material, such as autologous bone or bone substitute (e.g. Geistlich Bio-Oss®). Such defects should not be overfilled.
- > Geistlich Mucograft® is cut to the required size and shape with surgical scissors. The use of a template (e.g. sterile aluminum foil) may be helpful.
- > The compact layer should face upwards, away from underlying bone, and the spongy surface should face toward the bone.

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- › Complete penetration of the matrix by blood and exudates allows perfect adaptation and adhesion of the matrix to the underlying surface and facilitates the formation of a blood clot. To achieve adhesion the matrix is applied over the prepared site and held carefully in place. The period of time necessary will vary with the degree of bleeding.
- › Due to the high tensile strength of this matrix, fixation or suturing is possible. Fixation or suturing of the matrix may be indicated, depending upon the nature of the particular defect and to avoid matrix displacement. The matrix can be used both in closed and in open healing situations.

Special instructions for use in Periodontology

A basic requirement for successful periodontal treatment includes eradicating the underlying bacterial infection as well as adequate oral hygiene. Therefore, prior to surgical

intervention, patients must receive a hygiene phase of treatment, consisting of oral hygiene instructions, scaling and root planing, and occlusal adjustment when indicated. A post-operative maintenance phase can help to ensure long-term therapeutic success.

In order to avoid the formation of a long junctional epithelium, Geistlich Mucograft® must be adapted closely to the tooth (if necessary with additional fixation using suture material).

Post-operative Care

The usual post-operative care and medication should be given. Further prosthetic treatment should only be carried out after a healing period, to ensure complete soft tissue regeneration.

In case of bacterial contamination, rinsing with appropriate bactericidal solutions is recommended.

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In the event that early matrix removal is necessary, the tissues adjacent to the matrix should be anesthetised with a local anesthetic. An incision should then be made immediately adjacent to the residual matrix. Following careful reflection of the surrounding tissue, the remaining portion of the matrix can be excised and the area debrided to remove any inflamed or infected tissue.

Limitations of use

Contraindications

- > Geistlich Mucograft® should not be used in the presence of infected wounds.
- > Geistlich Mucograft® should not be used in patients with a known allergy to collagen.

Precautions

Geistlich Mucograft® should only be used by trained dentists and surgeons.
Geistlich Mucograft® should be used with special caution in patients with

- > acute or chronic infection (e.g. osteomyelitis) at the surgical site
- > uncontrolled metabolic diseases, e.g. diabetes, osteomalacia, thyroid disorder
- > prolonged corticosteroid therapy
- > autoimmune diseases
- > radiotherapy
- > heavy smoking

Due to the adherence to the underlying surface and the elasticity of Geistlich Mucograft®, the use of bone augmentation material is recommended to create and maintain space for bone regeneration in GTR procedures.

Side effects

As the matrix is a collagen product, in very rare cases allergic reactions may occur.
Due to the special physical properties and the prolonged absorption time an inflammatory reaction may occur in very rare cases.
Possible complications which may occur with any surgery include swelling at the surgical site,

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flap sloughing, bleeding, local inflammation, bone loss, infection, or pain.

Pregnancy / Lactation

Data on the use of the product during pregnancy or lactation are not available. For safety reasons: women who are pregnant or breastfeeding should not be treated with Geistlich Mucograft®.

Other information

The content of the double-blister pack is intended for single use only.

Re-sterilisation is not possible, because the collagen material may denature when re-sterilised. Discard any unused material.

Do not use the product after the indicated expiry date.

Store in a dry place at room temperature (15–25 °C).

The product is sterile, if the packaging is unopened and undamaged. If the sterile packaging has been damaged, the product must not be used.

Presentation

Geistlich Mucograft® is packed in sterile double-blister packs:

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Bahrain

Gulf Pharmacy & General
Store
a division of Gulf
Corporation of Technology
P.O. Box 2576, Villa 390
New Zinj 330
Manama
Kingdom of Bahrain

Yousuf Mahmood Husain
W.L.L.
P.O. BOX 23
Bldg. 97, Block 328
Road 322
Salmaniya Ave.
Manama
Kingdom of Bahrain

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Estonia

Sirowa Tallinn AS
Salve 2C
111612 Tallinn
Estonia

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Hong Kong

HEAT Consultancy Co. Ltd.
 1912-13, The Centre
 99 Queen's Road Central
 Hong Kong

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E. Bridde ehf
 Holtasmári 1
 201 Kópavogur
 Iceland

India

LifeCare Devices Private
 Limited
 LifeCare House
 841, Mori Road
 Mahim West
 Mumbai – 400016
 India

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Dent-Med Materials B.V.
 Dorpsstraat 72
 1713 HK Obdam
 Nederland

New Zealand

Henry Schein Shalfoon
 23 William Pickering Drive
 Albany, Auckland 0632
 New Zealand

Norway

Unident AS
 Postboks 362
 3193 Horten
 Norge

Philippines

Spectrumed Inc.
 Unit 2001-B West Tower
 Philippine Stock Exchange
 Center
 Exchange Road, Ortigas
 Center
 Pasig City 1605
 Philippines

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NOVODEX d.o.o.
Kneginje Zorke 2
11000 Beograd
Srbija

Singapore

Raydent supplies (S) PTE LTD
6 Ubi Road 1#04-01
Wintech Centre
Singapore 408726
Singapore

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Aurora Dental d.o.o.
 Brilejeva Ulica 2
 1000 Ljubljana
 Slovenija

South Africa

Implants for Africa (PTY) Ltd
 Unit 30 B
 Platinum Junction Business
 Park
 School Street
 7441 Milnerton
 South Afrika

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Vietnam

D.O.E.
 (Dentistry Of Excellence)
 199 Nguyen Thi Minh Khai
 Dist.1 – Ho Chi Minh City
 Vietnam

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