VOCO Traypurol®

EN Instructions for use MD EU Medical device

Concentrated cleaning agent for impression trays and instruments

Product description:

Traypurol removes residues of alginates, zinc oxide eugenol, carboxylate and phosphate cement from impression trays and instruments without using ultrasonic apparatus. **Traypurol** is biodegradable and provides gentle cleaning action.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Traypurol should only be applied by a professionally trained dental practitioner.

Use:

Use **Traypurol** in a 5 to 10 % solution (dilute approx. 50 - 100 ml **Traypurol** in 1 l water). For persistent residues, **Traypurol** can also be used in a more concentrated solution. Warm water accelerates the cleaning process. Pre-clean the trays and instruments and place them in the **Traypurol** solution. An upright position of the impression trays in the solution will enhance the cleaning process.

Residues are softened in the **Traypurol** solution within a few hours and can then be easily rinsed off under running water or removed with a brush. If possible leave impression trays and instruments soaking overnight in the solution.

Traypurol is suitable for all non-corroding metals and plastics. Renew the solution at least after 2 - 3 days. **Traypurol** is also very suitable for use in ultrasonic apparatus.

Warnings, precautionary measures:

- Safety glasses and gloves should be worn while using this product.
- In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:

Store at temperatures of 4 °C to 23 °C. Do not use after the expiry date.

Disposal :

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Traypurol** must be reported to VOCO GmbH and the responsible authority.

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VC 60 002289 E1 0925 99 © by VOCO