28 strips - 14 day treatment  
for both upper and lower jaw

**(EN)** Teeth whitening strips. Contains hydrogen peroxide. May not be used for patients under 18. Only to be sold to dental practitioners. For each cycle of use, the first usage is to be carried out by a dental practitioner or under their direct supervision if an equivalent level of safety can be guaranteed. May thereafter be handed to the consumer for continued treatment. Treatment duration: 1-2 hours.

Do not eat, drink or smoke while you whiten your teeth. Rinse your mouth thoroughly after treatment to remove any gel residue. Discard used strips.

**Warning:** Not recommended for pregnant women. Do not swallow strips. Avoid contact with eyes, if contact occurs, rinse with water immediately. Keep away from children.

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3

### Tip for best result!

Brush your teeth with WHITENING<sup>boost</sup> TOOTHPASTE from Brilliant Smile Sweden before commencing treatment. The toothpaste effectively removes coatings and surface stains, making the teeth whitening treatment more efficient.

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(EN)

**Instructions for use**

Whitening strips for whitening of discolored teeth. Recommended treatment duration: 1-2 hours/day.

**Fields of Application**

- Teeth whitening vital, discolored teeth
- Teeth whitening non-vital teeth
- Whitening of teeth discolored by medication

**Contraindications**

- Known allergy or intolerance to any of the ingredients
- Pregnancy and lactation

**Dental fillings and crowns, bridges and veneers made of composite or metal are not affected.** These can be replaced if needed to match the new whiter teeth. It is advisable to wait for a period of two weeks before such a replacement or restoration.

**After treatment**

During treatment and 72 hours after, avoid acidic foods and drinks (cola, citrus drinks and fruit), as these can cause hypersensitivity. Strong colored drinks (coffee, tea, red wine) and food (chocolate, tomatoes, curry, oranges, etc.), and tobacco should also be avoided during the same period of time to avoid discoloring of the teeth. Noticeably whiter teeth can usually be seen after approx. 1 week. For persistent stains, the Brilliant Smile certified staff may extend the treatment period further. Depending on the tooth structure and personal habits (consumption of discoloring food, smoking), the result can last for several years. By using Brilliant Smile Whitening toothpaste and mouthrinse, you may extend duration of the results even longer. Treatment may be repeated at a later time without causing any damage to the tooth enamel.

**Potential side effects**

In some cases the teeth may become slightly sensitive to hot and cold during treatment and a couple of days after treatment is completed, manifesting in shooting pains and hypersensitivity. Temporary gingival irritation during treatment may also occur. Please consult Brilliant Smile certified staff if problems persist. White spots on the teeth's surface may temporarily become more visible during treatment and a couple of weeks after. These are caused by transient dehydration.

**Warning**

Avoid contact with eyes, skin and mucosal tissue. In case of contact, rinse with water. If contact with the eyes occurs, you should consult a doctor as well.

**Storage**

Do not freeze. Protect from heat and direct sunlight. Do not use after the expiration date. Keep away from children!

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