#### Notes on application:

Please observe instructions for use and the maximum filling height of the device manufacturer. Not suitable for the removal of deposits or concretions in gingival pockets. Furthermore, do not direct the powder stream on the gums or other soft tissue. Exposed dentine and enamel caries must not be treated, as higher abrasive values can be expected. Composite, dental compomer, amalgam and gold fillings must not be treated either, as doing so could cause roughening. Patients who suffer from chronic bronchitis or asthma, should never be treated with an air polishing system. Patients with a low-sodium diet (e.g. high blood pressure) should not be treated with the powder, as it contains sodium hydrogen carbonate. Patients who are suspected of being allergic to the flavourings, must not be treated with the powder. Patients and operators must wear safety goggles. The operator must wear a mask and gloves. Use a high-performance suction device. Apply Vaseline to the patient's lips. Only for use in dentistry. Keep the containers closed. **Shake well before use!**

*All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the State in which the user/patient is established.*

# POLO DENT®

P O L I S H I N G

## Prophylaxis Powder Classic

### SUPRAGINGIVAL

- en** Prophylaxis powder supragingival
- nl** Profylaxepoeder supragingivaal
- fr** Poudre prophylactique supragingivale
- de** Prophylaxepulver supragingival
- pl** Proszek profilaktyczny do leczenia okolicy naddziąsłowej
- da** Profylaktisk pulver supragingival
- sv** Profylaxpulver supragingival
- no** Profylaksepulver supragingival
- fi** Profylaktinen jauhe ikenen yläpuoliseen puhdistamiseen

300 g e

**Type of application:** Sodium hydrogen carbonate-based prophylactic powder for the supragingival removal of intractable plaque and discolorations on enamel. Suitable for all commercially available air polishing systems.

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