

# ■ Foam 2 Wipes Unisepta®



Ready-to-use wipes impregnated with a cleaning and disinfecting solution for treating non-immersible and non-invasive medical devices (examples: dental unit, dental curing lamp body, etc.).

## Key product features

- Impregnation solution formulated without alcohol
- Contact time: 2 - 5 minutes
- Made from viscose - 100% biodegradable
- Without perfume or allergy promoting substances
- Ideal for treating the dental chair
- Wipe size: 180 x 200mm
- Convenient 100 wipes dispenser pack

# ■ Foam 2 Wipes Unisepta®

## Protocol for use

### Before use

Ensure that appropriate gloves are worn.



1 Remove the wipe from the pack.



2 Apply the wipe to the area to be treated.



3 Respect the contact time indicated for the desired antimicrobial activity. Rinsing is not necessary unless the medical device comes into contact with skin or mucous membranes.




4 Ensure that the pack is closed securely after each use.

### Precautions

Dangerous: respect the precautions for use (established according to the European regulations in force for classification and labelling of chemicals). Before each use, read the label and the information concerning the product. Storage between +5°C and +25°C. Reserved for professional use. Medical device CE 0459.

## Efficacy\*\*

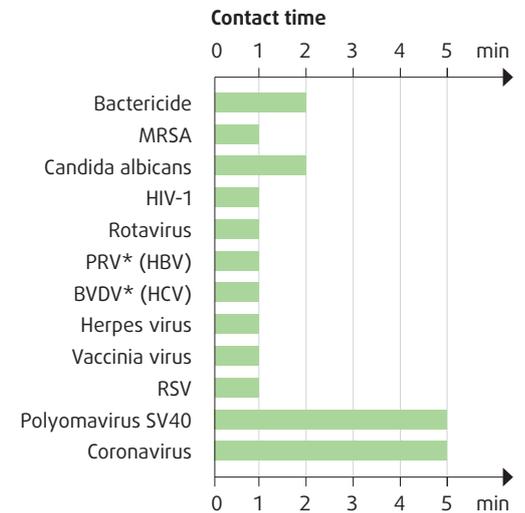

\* PRV: Surrogate of Hepatitis B  
\* BVDV: Surrogate of Hepatitis C  
\*\* Efficacy of impregnation solution

The Robert Koch Institute recommends the use of disinfectants claiming limited virucidal activity to interrupt the infection-chain. According to the RKI, limited virucidal activity (effective against enveloped viruses) corresponds to an activity against Vaccinia virus and BVDV.

### Conforms to European standards

The impregnation solution is:

Bactericide: EN 13727, EN 16615

MRSA: EN 13727

Levuricide: EN 13624, EN 16615

Active on viruses according to EN 14476

### Product composition

100g of impregnation solution contains: 0.30g of didecyltrimethylammonium chloride, excipients.

### Physical data

Wipe made from viscose (spunlace 45gr/m<sup>2</sup>), 100% biodegradable.

Colour: White wipe

Perfume: Without perfume

Shelf life: 2 years (unopened)

### Packaging

	100 Wipes	150 Wipes
Pack	✓	