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ENGLISH

## Composition

Geistlich Mucograft® / Geistlich Mucograft® Seal is a resorbable collagen matrix obtained by standardized, controlled manufacturing processes. The collagen is obtained from veterinary certified pigs and is carefully purified to minimize the risk of immunological reactions. Geistlich Mucograft® / Geistlich Mucograft® Seal is sterilized by  $\gamma$ -irradiation.   
Geistlich Mucograft® / Geistlich Mucograft® Seal has two structures, which form a bilayer product. The dense structure consists of compact collagen and has a smooth texture. A second structure consists of a thick, porous collagen spongy scaffold.

## Properties / Action

The low immunogenicity and proven biocompatibility favor the use of Geistlich Mucograft® / Geistlich Mucograft® Seal in oral surgeries. The long-fibred microstructure of Geistlich Mucograft® / Geistlich Mucograft® Seal readily absorbs fluid. The coherent collagen fibers of Geistlich Mucograft® / Geistlich Mucograft® Seal form a basic tissue matrix, which adheres well to the surrounding and underlying tissues. The dense structure of compact collagen protects the wound in open healing situations and allows tissue adherence and favorable wound healing. The second structure, of porous collagen, is placed next to the host tissue to support formation of the blood clot allowing formation of new blood vessels for wound healing. The degradation of Geistlich Mucograft® / Geistlich Mucograft® Seal takes several months depending on the surgical site, vascularization and patient comorbidities. Due to the natural origin of the collagen a slight bulging and variations in the thickness of Geistlich Mucograft® / Geistlich Mucograft® Seal matrix may be noted. Fixation with sutures or pins is possible. The matrix is not radio-opaque.

## Intended use

Geistlich Mucograft® / Geistlich Mucograft® Seal is intended to be used in closed and open healing situations to:

- > close wounds of oral mucosa
- > support wound healing and regeneration processes in case of oral mucosa defects and deficiencies.

The concerned oral mucosa defects and deficiencies may be surgically created or result from traumatic injuries, pathological conditions, medical treatments and therapies, or history of personal lifestyle habits.

## Indications

Geistlich Mucograft® / Geistlich Mucograft® Seal is indicated to be used in the surgical treatment of the following clinical conditions:

- > defects and deficiencies of oral mucosa associated with:
  - a) presence of recessions
  - b) lack of keratinized tissue
  - c) scar tissue.
- > oral mucosa wounds associated with:
  - a) bone regeneration procedures
  - b) lack of soft tissue.

## Instructions for Use

Geistlich Mucograft® / Geistlich Mucograft® Seal are to be used by licensed dentists/physicians trained in a surgical management of oral soft tissue. The general principles of sterile handling and patient medication must be followed when using Geistlich Mucograft® / Geistlich Mucograft® Seal.

- > The defect is exposed by means of (a) properly prepared flap(s) or tooth/teeth extraction and the usual surgical procedures in creating a properly prepared site are undertaken.
- > Geistlich Mucograft® / Geistlich Mucograft® Seal is cut if necessary to the required size and shape.
- > Geistlich Mucograft® / Geistlich Mucograft® Seal must remain dry during trimming and application to the defect. Geistlich Mucograft® Seal is designed to support mucosa wound healing after tooth extraction.
- > The Geistlich Mucograft® Seal spongy structure is identified by an indented stripe to assure the correct side is placed facing the bone. The compact structure must face outward, away from underlying bone. To achieve adhesion, the matrix is applied over the prepared site and held carefully in place for a period to allow penetration of the matrix by blood and exudates, which allows good adaptation and adhesion of the matrix to the underlying surface and facilitates the formation of a blood clot.
- > No compression of Geistlich Mucograft® / Geistlich Mucograft® Seal: The collagen matrix should remain uncompressed before, during and after surgery.
- > No tension around Geistlich Mucograft®: Any tension of the soft tissues around Geistlich Mucograft® should be avoided.
- > When using Geistlich Mucograft® Seal in bone regeneration procedures, the underlying bone deficiency must be filled with a space-maintaining material (e.g. Geistlich Bio-Oss® Collagen).

- > Geistlich Mucograft® / Geistlich Mucograft® Seal can be used both in closed and in open healing situations. When used for treatment of deficiencies of the oral mucosa associated with presence of recessions, Geistlich Mucograft® should be used completely submerged.
- > Local use of Geistlich Mucograft® / Geistlich Mucograft® Seal with medicinal products, alcohol, disinfectants or antibiotics has not been studied.

### Special instructions for use in periodontology

A basic requirement for successful periodontal treatment includes eradicating the underlying bacterial infection as well as adequate oral hygiene. Therefore, prior to surgical intervention, patients must receive a hygiene phase - treatment, consisting of oral hygiene instructions, scaling and root planing, and occlusal adjustment when indicated.

### Post-operative care

The usual post-operative care and medication should be given. Further prosthetic treatment should only be carried out after a healing period, to ensure complete soft-tissue regeneration.

### Limitations of use

#### Contraindications

- > Geistlich Mucograft® / Geistlich Mucograft® Seal should not be used in the presence of acutely infected wounds.
- > Geistlich Mucograft® / Geistlich Mucograft® Seal should not be used in patients with a known allergy to collagen.

#### Precautions

Geistlich Mucograft® / Geistlich Mucograft® Seal should be used with special caution in patients with:

- > chronic infection (e.g. osteomyelitis) at the surgical site
- > uncontrolled metabolic diseases, e.g. diabetes, osteomalacia, thyroid disorder
- > prolonged corticosteroid therapy
- > autoimmune diseases
- > radiotherapy
- > heavy smoking.

Geistlich Mucograft® / Geistlich Mucograft® Seal is a typical non-metallic diamagnetic material, in common with most human tissues. It cannot be heated or act as a magnet during the MRI examination. Geistlich Mucograft® / Geistlich Mucograft® Seal has not been specifically studied in the MR environment.

Geistlich Mucograft® / Geistlich Mucograft® Seal may be used with metal hardware. In this case, special caution under the MR environment may be required.

Intraoperatively, if there is need to remove the product, complete removal can be achieved. In the post-operative phase, complete removal may not be possible since the product is integrated over time with the existing tissue.

The grafted area should not be brushed for the first four weeks and excessive muscle tractioning or trauma to the treated areas should be avoided for the first four weeks.

The product is intended for use in a single patient in a single medical procedure. Any product not required in the single procedure should be discarded since sterility and stability cannot be assured once the original primary packaging is opened.

### Side effects

As the matrix is a collagen product, in very rare cases allergic reactions may occur. Possible side effects that may occur with any surgery include: swelling at the surgical site, flap sloughing, bleeding, local inflammation/necrosis, and infection. These side effects could lead to reduced tissue healing. Other surgery-related disturbances that may appear include bone loss and pain.

### Pregnancy / lactation

Data on the use of the product during pregnancy or lactation are not available.

### Other information

-  Do not re-use
-  Do not re-sterilize
-  Use-by date
-  Temperature limit
-  Keep dry
-  Do not use if package is damaged
-  Keep away from sunlight
-  Caution
-  Consult electronic instructions for use
-  Sterilized using irradiation
-  Batch code
-  Catalogue number
-  Recycling symbol
-  Russian import symbol
-  Ukrainian registration symbol

 Authorized representative in the European Community / European Union

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Geistlich Pharma AG