Curing Light **Pro**

FOR COMPOSITE POLYMERISATION

INSTRUCTIONS FOR USE

POLO|DENT[®]

Curing Light **Pro**

FOR COMPOSITE POLYMERISATION

en	Instructions For Use	4
nl	Gebruiksaanwijzing	8
fr	Mode d'emploi.....	12
de	Gebrauchsanweisung	16
pl	Instrukcje użytkowania.....	20
da	Brugsanvisning	24
sv	Bruksanvisning	28
no	Instruksjoner for bruk	32
fi	Käyttöohjeet	36
es	Instrucciones de uso.....	40
pt	Instruções de utilização.....	44
it	Istruzioni per l'uso	48
hr	Upute za upotrebu	52
sl	Navodila za uporabo	56
sk	Návod na použitie	60
cs	Návod k použití.....	64

Curing Light Pro

Please read this manual before operating

Safety Precautions

1. This product is intended for use in hospitals and dental clinics only. The use of the product must comply with the requirements of the relevant operating specifications and relevant regulations of the medical department. Users must be professionally trained and qualified as dentists or technicians. Operators are always suggested wearing protectors (safety goggles, face shield...) to avoid infection.
2. The products are reusable:
Before each use, place a new disposable sleeve over the curing light's light tip to avoid the direct contact between the curing light and the patient's skin or oral mucosa, then to avoid cross-infection. After use, remove the disposable sleeve and dispose of it according to relevant regulations. Manually clean and disinfect the surface of the light tip with clean water or disinfectant. Do not soak the light tip. The disposable sleeve must meet the requirements of relevant medical device regulations. No reuse of disposable sleeve to avoid cross-infection!
3. During the operation, the light source should be irradiated straightly at the composite resin to ensure the effect of solidification.
4. Please use the original protective light shield and follow the right installation and application method to avoid the damage of blue-violet light to the eyes.
5. Please do not use other than the specified voltage to recharge the battery. Only the original charging base, adapter and lithium battery could be used, because other brand charging base, adapter and lithium battery may cause the damage of the circuit.
6. Please recharge the battery at least 4 hours before first time usage. It's forbidden to connect the adapter to the base for long time while not charging and it's not allowed to use the curing light while charging.
7. It is forbidden to insert the metal or other conductor into the charging connector, to avoid damaging the circuit or the battery.
8. Please recharge the battery in cool and ventilated room. Please press the main unit to the buckle of charging base, in order to avoid the poor connection.
9. It is forbidden to self-taking-apart the battery, in order not to result in short-circuit or leakage.
10. It is forbidden to extrude or shake the battery. The lithium battery is forbidden to be put in short-circuiting situation like putting the battery with metal or other conductors.
11. The instrument has electromagnetic interference. Do not use it around electronic surgery, and use the instrument with caution in an environment with strong electromagnetic interference.
12. This product does not contain toxic or hazardous substances and it is discarded in accordance with the relevant laws and regulations on waste medical devices. Please deal with the replaced battery according to national regulations.
13. As a professional manufacturer of medical instruments, we are only responsible for the safety on the following conditions:
 - a. The maintenance, repair and modification are made by the manufacturer or the authorized dealer.
 - b. The changed components are original of our company and operated correctly according to instruction manual.

Product contraindications:

The heart disease patient, pregnant woman, infants and the person who are allergic to blue-violet light should be cautious to use this equipment.

Symbol / Instruction

Symbol	Instruction	Symbol	Instruction
	Caution		Refer to instruction manual/booklet
	Date of manufacture		Manufacturer
	Class II equipment		Type B applied part
	For indoor use only		Recovery
	Keep dry		Fragile, handle with care
	Atmospheric pressure limitation 70kPa ~ 106kPa		Temperature limit -20°C ~ +55°C
	Humidity limitation 10%~93%		Waste electrical and electronic equipment
	CE Mark		On/off
	Authorized Representative in the EUROPEAN COMMUNITY		Serial number
	Medical Device		Unique device identifier
	Mode/Time		Reference number

1. Product Introduction

The curing light uses the principle of light radiation to irradiate photosensitive resin in order to quickly cure it. This product is intended for use in hospitals and dental clinics and it's used for prevention and treatment of dental complications of adults or children.


Scope of application:

This curing light is intended for use by trained dental professionals for the purpose of light curing dental resins and composites.

Product composition: main unit, light tip, protective light shield, charging base, battery and adapter, etc.

The features of this product are:

- 1.1 A variety of different working modes and working time can be set.
- 1.2 Constant light power output, the decrease of battery power will not affect the curing effect.
- 1.3 Large-capacity battery. When full charged, under 10 seconds of Mode Normal, it can be used continuously for more than 800 times.
- 1.4 Product schematic.

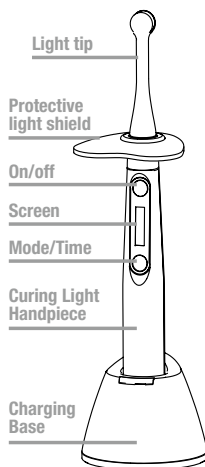2. Replacement instructions for accessories

Accessories	Period of replacement	Method of replacement
Power adapter	Damaged	Replace directly according to the requirements of the manual.
Charging Base	Damaged	Replace directly according to the requirements of the manual.
Battery	Damaged	Take out the damaged battery and replace it with a new one.
Protective light shield	Damaged	Replace directly according to the requirements of the manual.

3. Technical parameters

3.1 Technical Parameters

Model	PoloCuring Light Pro
Dimensions	Ø72 mm x 250 mm
Net weight	215 g
Classified by power supply	Powered by a rechargeable lithium battery
Input	Lithium battery 3.7V 1900 mAh
Rechargeable lithium battery	Battery model: ICR 18500, Battery voltage and capacity: 3.7V/1900mAh. The battery has overvoltage, overcurrent and short circuit protection
Power supply (Adapter)	Input: 100-240V a.c, 50Hz/60Hz, 0.4A Max Output: 5.0V d.c, 1.0A
Performance of led lamp	a) 5W high power blue and purple LED lamp. b) Wavelength: 385nm-515nm c) Category: class I. d) Shoot limit (AEL): $3.9 \times 10^{-3} \text{J}$
Light intensity	1000 mW/cm ² ~ 3000 mW/cm ²
Inspected method of LED lamp	When operating correctly, the LED lamp lights up, indicating that the lamp is in good condition. Dental resin materials commonly used in clinic can match the wavelength of this curing light, such as 3M, Dentsply and other resins.
Modes and time setting	1. Turbo: Blue and violet light, 2700-3000 mW/cm ² for 1s, 2s, 3s; 2. High: Blue and violet light, 2300-2500 mW/cm ² for 1s, 3s, 5s; 3. Normal: Blue and violet light, 1000-1200 mW/cm ² for 5s, 10s, 15s, 20s; 4. Ortho: Blue and violet light, 2300-2500 mW/cm ² for 3s to work 5 times or 10 times every cycle, interval 1s; 5. Ramp: Blue and violet light, 1000-1200 mW/cm ² for 5s, 10s, 15s, 20s
Effective area of irradiation light	75.4 mm ²

3.2 Environmental factors

- 3.2.1 Operation temperature: +5°C ~ +40°C
- 3.2.2 Relative humidity: 10% ~ 93%
- 3.2.3 Atmospheric pressure: 70kPa~106kPa

3.3 Equipment safety classification

- 3.3.1 Classified by type of protection against electric shock: Class II
- 3.3.2 Classified by degree of protection against electric shock: Type B application part
- 3.3.3 Classified according to the degree of protection against incoming liquid: Ordinary equipment (IPX0), non-waterproof
- 3.3.4 Classified by operation mode: Non-continuous operation
- 3.3.5 Classification of safety degree when using flammable anesthetic gas mixed with air or oxygen or nitrous oxide: Equipment that cannot be used with flammable anesthetic gas mixed with air or oxygen or nitrous oxide.

4. Installation and demounting

- 4.1 Align the large end of the light tip with the mounting hole on the upper end of the curing light machine and insert it (it must be inserted to the bottom by rotating it, do not insert it at a single angle directly).
- 4.2 Put the disposable sleeve on the light tip, and install the protective light shield plate tightly at the bottom of the light tip.
- 4.3 When disassembling or replacing the light tip, proceed in the reverse direction above.

5. Operation

- 5.1 When powered by battery, press the power button(on/off) to wake up the curing light in sleep mode.
- 5.2 Mode setting: Long press the M/T button to select the mode. Refer to the Table 1 for the settable modes.
- 5.3 Time setting: Short press the M/T button to select the time. Refer to the Table 1 for the settable curing time.
- 5.4 Then aim the light source at the position that needs to be cured, press the power button, the host will make a "beep" sound, the host will work according to the selected working mode, and the display will count down from the selected timing time. After the timer ends, the display shows the set timing time.
- 5.5 Before the normal use time is over, press the power button to end the work and it can end the output at any time.
- 5.6 After one irradiation cycle is over, you can immediately press the power button to start the next irradiation cycle. If the curing light continuously for 60 seconds under a light intensity greater than 1200 mW/cm², you need to wait for the surface temperature of the metal head to return to normal temperature then it can continue to be used.
- 5.7 After use, if the transparent cover is stained with resin, please wipe it clean with cotton cloth to avoid affecting the light intensity.
- 5.8 The effective light intensity of this machine is many times higher than that of the halogen light curing machine, and the curing depth of light-cured composite resin isn't less than 4mm in 10 seconds.
- 5.9 The light tip at the front end can be removed, but it cannot be sterilized by high temperature and high pressure, otherwise it will cause damage.
- 5.10 Select the Normal mode for 10 seconds. Then pressing the power button, the output will start. When outputting, immediately press the power button holding for 2 seconds to switch Chinese or English; Select the Normal mode for 15 seconds. Then pressing the power button, the output will start. When outputting, immediately press the power button holding for 2 seconds to switch the left handed or right handed operation.

6. Low battery prompt and charging

- 6.1 Low voltage indication: The battery symbol on the display starts to flash.
- 6.2 Put the main unit correctly into the charging connector of the charging base, and the screen of the main unit will display the charging status.

7. Transportation, storage and maintenance

7.1 Transportation

- 7.1.1 During transportation, avoid shocking and vibration, handle with care and avoid inversion.
- 7.1.2 Do not mix with dangerous goods during transportation.
- 7.1.3 Avoid sunlight, rain or snow during transportation.

7.2 Storage

- 7.2.1 Do not mix with toxic, corrosive, flammable and explosive materials during storage.
- 7.2.2 The product should be stored in an environment where the relative humidity does not exceed 10% ~ 93%, the atmospheric pressure is 70kPa ~ 106kPa, and the temperature is -20°C ~ +55°C.

7.3 Maintenance

- 7.3.1 This equipment should be handled with care, far away from the source of the earthquake, and should be installed or stored in a cool, dry and ventilated place.
- 7.3.2 When the device is not in use, turn off the curing light and unplug the adapter.
- 7.3.3 To clean or disinfect, wipe the surface with clean water or disinfectant, do not soak.
- 7.3.4 After use, check if there is any resin remaining on the surface of lamp, so as not to affect the life of the light tip or affect the curing effect.

8. Troubleshooting

Faulty	Possible cause	Solutions
No reaction	1. Out of battery 2. Faulty of battery 3. Battery protection	1. Charge/Change a new battery 2. Change a new battery 3. Activate it by putting it on the base
Insufficient Light intensity	There is resin remaining on the surface of the light tip	Remove the resin
Not charging when connecting with the adapter	1. The adapter is not connected well 2. Faulty of adapter or incompatible 3. There is impurity on the connector of charging base	1. Reconnect 2. Change the adapter 3. Clean the inner connector of charging base by the alcohol
Less working life after fully charge	Capacity of battery decreased	Change a new battery
When the battery icon flickering on the screen of main unit	Low voltage	Back to normal after 15-minute charging.

If the fault still cannot be resolved, please contact your local dealer or our company.

9. Electromagnetic compatibility

Attention:

- This Curing Light meets the relevant requirements of IEC 60601 standard electromagnetic compatibility. The basic performance of the light curing machine is: the front LED light can emit blue & violet light when working.
- The user should install and use it according to the electromagnetic compatibility information provided by the accompanying documents.
- Portable and mobile radio frequency communication equipment may affect the performance of the curing light, avoid strong electromagnetic interference when using it, such as close to mobile phones, microwave ovens, etc.;
- The guidelines and manufacturer's declaration are detailed in the attachment.
- Without the express consent of manufacturer unauthorized changes or modifications to the equipment.
- it may cause damage to the equipment or other problems of electromagnetic compatibility to the equipment.

Alert:

- This light curing machine should not be used close to or stacked with other equipment. If it must be used close or stacked, it should be observed and verified that it can be used in the configuration used.
- Except for the cables sold by the manufacturer of the curing light as spare parts of internal components, the use of accessories and cables other than those specified may cause the curing light increase in emission or decrease in immunity.
- The following cables must be used to comply with electromagnetic emission and anti- interference requirements:

Cable Name	Cable Type	Cable Length
Power Line (Output end)	Unshielded Parallel Lines	1.2 m

- Key components of electromagnetic compatibility

The key components of electromagnetic compatibility of this product are IC chip, main PCB, lithium battery, power adapter and LED lamp. Use or replace the accessories and cables that are not designed to match, will cause the electromagnetic compatibility emission and anti-interference performance to be significantly reduced. Do not change machine parts without authorization.

10. Environmental protection

Please dispose according to the local laws.

11. After sales service

- 11.1 If this equipment cannot work normally due to quality problems from the date of sale, please show the warranty card and our company will be responsible for the maintenance. Please refer to the product warranty card for the warranty period and warranty scope.
- 11.2 This product does not contain replaceable parts. All maintenance, adjustment, calibration and modification of technical parameters of the product can only be carried out by our company's technicians or special repair shops. If customers need to repair themselves, our company can provide them on request Circuit diagrams, component lists, legends, calibration specifications, or other materials needed to help qualified technical personnel repair the equipment parts designated by our company, but we will not bear any consequences.
- 11.3 The user must use the original parts, purchase please contact with the local dealer or the manufacturer. Do not use the related accessories of other brands, so as not to cause damage to the light curing machine or other dangers.
- 11.4 After the power adapter, charging base and other accessories are damaged, users should not repair by themselves. Please buy new parts and replace them. If any relevant information needed, please contact the manufacturer.

12. Special instructions

The company reserves the right to modify the machine design, product technology or accessories, operation manual and machine packaging content at any time without prior notice. The product is subject to the real object, and the picture is for reference only. The final interpretation right belongs to the manufacturer.

Product life: 10 years

Production date: please refer to packaging label.

 **Polo MB Oisterwijk BV**
Laarakkenweg 5
5061 JR Oisterwijk, The Netherlands

 **Gullin Mecco Medical Instrument Co., Ltd.**
3rd Floor No.8, Information Industrial Park, High-Tech Zone,
Qixing District, Guilin, Guangxi, 541004, P.R. China

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**MedNet EC-REP GmbH**