

## 3M post and core solution. Featuring 3M™ RelyX™ Fiber Post 3D.

#### For predictable results – in less time.

We know: The endodontic procedure is very complex. So the simpler and more reliable the endodontic restorative procedure is, the better – for both the dentist and the patient. The excellent 3M post and core solution is a big step forward in this regard: With significantly fewer clinical steps and fewer products, it lets you work faster, easier and more efficiently.

#### All you need to do is take four steps.

The 3M post and core solution not only simplifies your work but also helps you deliver predictable, robust outcomes – from root to core. All you need is the 3M™ RelyX™ Fiber Post 3D Glass Fiber Post plus three other easy-to-use products from 3M Oral Care: 3M™ RelyX™ Unicem 2 Automix Cement, 3M™ Scotchbond™ Universal Adhesive, and 3M™ Filtek™ One Bulk Fill Restorative.

These four products – each of them proven to make your work more reliable and efficient – have one common objective: Simplify the complex!

# Fewer clinical steps, fewer products.




### **Post**

## 3M™ RelyX™ Fiber Post 3D Glass Fiber Post – With a unique 3D design for more confidence

RelyX Fiber Post 3D builds on the 8 years of clinical success of the proven product RelyX Fiber Post. Now, we have further improved it to enhance your peace of mind:

- High mechanical retention to the core build-up material thanks to coronal 3D macro retention.
- Strong bond in the root canal due to its microporous surface without a silanating step.
- Better post position control thanks to excellent radiopacity.
- Low risk of root fracture due to dentin-like elasticity.
- More esthetic option than a metal post.

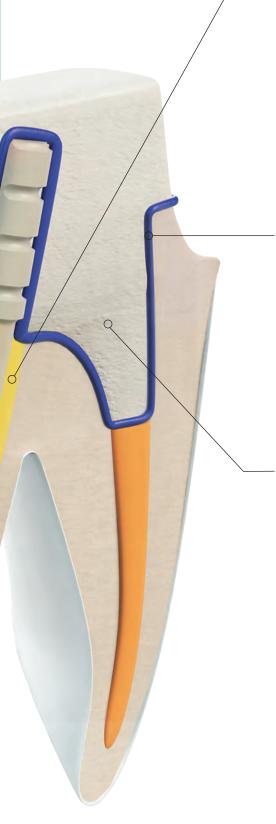

## Cement



#### 3M<sup>™</sup> RelyX<sup>™</sup> Unicem 2 Automix Self-Adhesive Universal Resin Cement

RelyX Unicem 2 Automix Cement not only offers excellent bond strength but also saves you precious time, as you do not need to etch, prime or bond.

The handy automix syringe, together with the endo tip, lets you apply the ready-mixed cement directly in the root canal from bottom to top.

With one application - and virtually no voids.

## **Adhesive**

#### 3M™ Scotchbond™ Universal Adhesive

Scotchbond Universal Adhesive provides a reliable bond of the core build-up material to the post and tooth structure – with no need for an additional primer.

One application – done!



## Core build-up

#### 3M™ Filtek™ One Bulk Fill Restorative

Excellent material and handling characteristics make Filtek One Bulk Fill Restorative a great choice of core build-up material. With one-step increments up to 5mm, core build-ups can be done faster and easier.



3M™ RelyX™ Fiber Post 3D Glass Fiber Posts are available in four different sizes. Color codes ensure an accurate match to the drills.





#### **Ordering Information**

Itam Na	Product Information		
	rY™ Fiber Post 3D Glass Fiber Post		
56957 Introductory Kit			
00007	20 posts sizes 0, 1, 2, 3 (5 posts of each size), 5 drills sizes 0, 1, 2, 3, Universal (1 drill of each size)		
56958	Trial Kit with 3M™ RelyX™ Unicem 2 Automix Cement 10 posts sizes 1, 2 (5 posts of each size), 3 drills sizes 1, 2, Universal (1 drill of each size), 1 – 3M™ RelyX™ Unicem 2 Automix Syringe A2 – 8.5 g, 10 mixing tips, 10 endo tips		
56960	Post and Core Build-up Kit with 3M™ RelyX™ Unicem 2 Automix Cement  1 Trial Kit 3M™ RelyX™ Fiber Post 3D / 3M™ RelyX™ Unicem 2 Automix Cement, 20 capsules 3M™ Filtek™ One Bulk Fill Restorative A3 – 0.2g each, 1 vial 3M™ Scotchbond™ Universal Adhesive – 5 ml		
Refills posts and drills			
56948	10 posts, size 0 (1.1 mm dia), white		
56949	10 posts, size 1 (1.3 mm dia), yellow		
56950	10 posts, size 2 (1.6 mm dia), red		
56951	10 posts, size 3 (1.9 mm dia), blue		
56873	1 drill, size 0 (1.1 mm dia), white		
56864	1 drill, size 1 (1.3 mm dia), yellow		
56865	1 drill, size 2 (1.6 mm dia), red		
56866	1 drill, size 3 (1.9 mm dia), blue		
56867	1 Universal drill		
3M <sup>™</sup> RelyX <sup>™</sup> Unicem 2 Automix Self-Adhesive Resin Cement			
	Refills		
56846	Shade A2 Universal  1 syringe – 8.5 g, 15 mixing tips (regular),  5 mixing tips (wide), 5 endo tips		
56847	Shade A3 Opaque 1 syringe – 8.5 g, 15 mixing tips (regular), 5 mixing tips (wide), 5 endo tips		
56848	Shade Translucent 1 syringe – 8.5 g, 15 mixing tips (regular), 5 mixing tips (wide), 5 endo tips		
	Value Packs		
56849	Shade A2 Universal 3 syringes – 8.5 g, 30 mixing tips (regular), 20 mixing tips (wide), 15 endo tips, 5 intra-oral tips		
56857	Shade A3 Opaque 3 syringes – 8.5 g, 30 mixing tips (regular), 20 mixing tips (wide), 15 endo tips, 5 intra-oral tips		
56858	Shade Translucent 3 syringes – 8.5 g, 30 mixing tips (regular), 20 mixing tips (wide), 15 endo tips, 5 intra-oral tips		
Accessories - Cement Mixing Tips			
56851	30 mixing tips (regular)		
56919	15 mixing tips (wide)		
56918	15 mixing tips (wide), 15 intra-oral tips		

Item No.	Product Information		
3M <sup>™</sup> Scotchbond <sup>™</sup> Universal Adhesive			
41254	Introductory Kit  1 vial 3M™ Scotchbond™ Universal Adhesive – 5 ml,  50 disposable mixing wells, 50 disposable microbrushes,  1 etchant syringe – 3 ml, 25 dispensing tips for etchant syringe		
41258	<b>Refill</b> 1 vial 3M <sup>™</sup> Scotchbond <sup>™</sup> Universal Adhesive – 5 ml		
3M™ Filtek™ One Bulk Fill Restorative			
Refill	Item No.	Item No.	
Shade	Syringe incl. 1 syringe – 4 g	Capsule incl. 20 capsules – 0.2 g each	
A1	4866A1	4867A1	
A2	4866A2	4867A2	
A3	4866A3	4867A3	
B1	4866B1	4867B1	
C2	4866C2	4867C2	
3M <sup>™</sup> Filte	k™ Bulk Fill Flowable Resto	rative	
Refill	Item No.	Item No.	
Shade	Syringe incl. 2 syringes – 2g each, 20 dispensing tips	Capsule incl. 15 capsules – 0.2g each	
A1	4862A1	4861A1	
A2	4862A2	4861A2	
A3	4862A3	4861A3	
Universal	4862U	4861U	
3740T	20 dispensing tips 19-gauge		
3M™ Retraction Capsule			
56941	Refill Pack: 25 capsules		
56942	Value Pack: 50 capsules		
3M™ Elipa	3M™ Elipar™ DeepCure-S LED Curing Light		
76975	1 stainless steel handpiece (cordless), charging base (230V), Li-ion battery, 10 mm light guide, eye shield		
3M™ Elipar™ DeepCure-L LED Curing Light			
76973	1 lightweight handpiece (built-in Li-ion battery), universal power supply with 5 adapters, 10 mm light guide, eye shield, 3 curing discs		



**3M Oral Care** 2510 Conway Avenue St. Paul, MN 55144-1000 USA 1-800-634-2249 3M Canada

Post Office Box 5757 London, Ontario N6A 4T1 Canada 1-888-363-3685

#### 3M.com/RelyX3D

3M, Elipar, Filtek, RelyX and Scotchbond are trademarks of 3M Company or 3M Deutschland GmbH. Used under license in Canada. All other trademarks are owned by other companies. Please recycle. Printed in U.S.A. © 3M 2017. All rights reserved. 70-2013-0728-0 (Rev 9/2017)