Prior to use, carefully read the instructions for use.

EN

CAVITON

HYDRAULIC TEMPORARY RESTORATIVE

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

As a temporary filling material.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

- Dry the prepared cavity with a small cotton pellet.
- Transfer the material to the cavity using a suitable instrument and form the contour. In the case of occlusal Class I cavities let the patient bite down on the surface, and then form the contour.
- Ask the patient to close his/her mouth. The filling will harden in about 30 minutes by absorbing moisture. Recommend the patient to avoid pressure or abrasion for at least 1 hour.

Notes

- Do not mix or contaminate with any other material.
- When removing the filling, clean the cavity with an alcohol soaked cotton pellet.
- Close the jar immediately.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F). (Shelf life: 2 years from date of manufacture)

PACKAGES

Jar 30 g, white or pink.

Last revised : 02/2015

MANUFACTURED by
GC ASAHİ CORP.
1-3-3 Kachigawa-cho, Kasugai, Aichi 486-0945, Japan

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

 EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC ASIA DENTAL PTE. LTD.
11 Tampines Concourse, #03-05, Singapore 528729
TEL: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Banksmeadow, NSW 2019, Australia
TEL: +61 2 9316 4499

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