

Pocket-X® Gel Case Selection Guide

Use this guide to identify ideal patients for Pocket-X® Gel as an adjunct to non-surgical periodontal therapy.

✓ Ideal Patient Candidates

- **Adults with diagnosed periodontitis**
Especially Stage 2–3 periodontitis where pockets remain after SRP
Pockets ≥ 5 mm with bleeding on probing
- **Patients scheduled for non-surgical periodontal therapy (RSD/SRP)**
Best placed at the end of hygienist sessions
- **Patients showing inflammation despite maintenance**
Red/swollen gums, Bleeding on brushing or probing, Chewing discomfort, Bad taste/halitosis
- **Patients motivated for gum health**
Likely to comply with aftercare (no eating/rinsing immediately, gentle brushing, avoiding interdental cleaning for 2 days)
- **Patients wishing to avoid or delay surgery**
Suitable for conservative treatment with less discomfort

✗ Exclusion Criteria

- Known allergy to hyaluronic acid, octenidine, phenoxyethanol, or poloxamer.
- Pregnant or breastfeeding women.
- Children or adolescents
- Patients unable to follow aftercare instructions.



Pocket-X® Gel is a patented Class IIa medical device. Apply after scaling and root planing as part of a non-surgical periodontal treatment program.

© Geistlich Pharma AG | www.Geistlich.com/dental