Porcelain Etch Gel

9,6% HYDROFLUORIC ACID

-  Porcelain etching gel syringes
-  Porselein etsgel syringes
-  Seringues d'Etching pr la porcelaine
-  Porzellan Ätzgel Spritzen
-  Porcelain æts gel i sprøjter

instructions for use

Porcelain etching gel syringes

Product description:

PoloDent Porcelain Etch Gel is an etching gel based on hydrofluoric acid for the extraoral etching of ceramic restorations.

Areas of application:

The adhesion between the ceramic surface and luting composites is increased by the production of retentive etching patterns. For all-porcelain restorations, i.e. onlays, inlays, crowns or veneers.

Instructions for use:

- Apply the etching gel to the porcelain surface with a small disposable brush or disposable application tip.
- The recommended reaction time is minimum 60 sec.
- Acid residue should be completely removed after the reaction time. The surface can be steam cleaned, rinsed with plenty of water or cleaned with distilled water in an ultrasonic cleaner to remove the acid residue. The porcelain surface should be etched immediately before fitting or after try-in. Avoid contaminating the etched surfaces. If surface is contaminated, it should be briefly re-etched (approx. 10 sec). Dry thoroughly, then apply bonding agent etc. according to the manufacturer's instructions.

Only intended for extraoral use. Must not be used to etch tooth structure intraorally!

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

Safety instructions:

- H310 - Fatal in contact with skin.
- H331 - Toxic if inhaled.
- H314 - Causes severe skin burns and eye damage.
- P280 - Wear protective gloves and eye protection.
- P261 - Avoid breathing dust/fume/gas/mist/vapours/spray.
- P301+P310 - IF SWALLOWED: Immediately call a POISON CENTER/doctor.
- P305+351+338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- P361 - Take off immediately all contaminated clothing.
- P321 - Specific treatment (see on this label).
- P501 - Dispose of contents/container to local / regional / national / international regulations.

Contains hydrofluoric acid. For professional use only.

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