EN	INSTRUCTIONS FOR USE .....	3
FR	INSTRUCTIONS D'UTILISATION .....	10
IT	ISTRUZIONI PER L'USO .....	17
DE	BENUTZERHANDBUCH .....	24
ES	INSTRUCCIONES DE USO .....	31
PT	INSTRUÇÕES DE UTILIZAÇÃO .....	38
NL	GEBRUIKSINSTRUCTIES .....	45
SV	BRUKSANVISNING .....	52
DA	BRUGSANVISNING .....	59
NO	BRUKSANVISNING .....	66
FI	KÄYTTÖOHJE .....	73
PL	INSTRUKCJA UŻYCIA .....	80
LT	NAUDOJIMO INSTRUKCIJA .....	87
LV	LIETOŠANAS NORĀDĪJUMI .....	94
ET	KASUTUSJUHISED .....	101
CS	NÁVOD K POUŽITÍ .....	108
SK	NÁVOD NA POUŽÍVANIE .....	115
HU	HASZNÁLATI ÚTMUTATÓ .....	122
RO	INSTRUCȚIUNI DE UTILIZARE .....	129
SR	UPUTSTVA ZA UPOTREBU .....	136
BS	UPUTSTVA ZA UPOTREBU .....	143
CNR	UPUTSTVA ZA UPOTREBU .....	150
SQ	UDHËZIMET E PËRDORIMIT .....	157
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ .....	164
TR	KULLANIM TALİMATLARI .....	171

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B00THPR230VEU / B00THPR230VUK / B00THPR115VUS

#### **1) INDICATIONS FOR USE**

The Thermaprep® Obturator Oven has been specially developed for heating the specified endodontic obturators brands for use in root canal therapy. Do not use this oven for any other purpose.

The Thermaprep® Obturator Oven is an accessory to specified obturators and will assist in achieving the performance characteristics of these medical devices. This product has to be used only in a clinical or hospital environment, by qualified users.

#### **2) CONTRAINDICATIONS**

There are none known.

### 3) COMPATIBLE DEVICES

The Thermanprep® Obturator Oven is compatible with the following endodontic obturator brands:

Obturator core	Obturator brand	Obturator size	Heat setting	Signal time	Tolerance
Gutta percha	GuttaCore®	20 to 90	Setting 1	20 seconds	± 5 Seconds
	GuttaCore® for WaveOne Gold™	Small - Primary - Medium - Large	Setting 1	20 seconds	± 5 Seconds
	GuttaCore® for ProTaper Next®	X2 / X3 / X4 / X5	Setting 1	20 seconds	± 5 Seconds
Plastic	Thermafil®	20 to 25	Setting 1	20 seconds	± 5 Seconds
		30 to 60	Setting 2	44 seconds	± 7 Seconds
		70 to 140	Setting 3	49 seconds	± 8 Seconds
	Thermafil® for WaveOne Gold™	Small - Primary	Setting 1	20 seconds	± 5 Seconds
		Medium - Large	Setting 2	44 Seconds	± 7 Seconds
	Thermafil® for ProTaper® Universal	F1 - F2	Setting 1	20 seconds	± 5 Seconds
		F3 - F4 - F5	Setting 2	44 Seconds	± 7 Seconds
	Thermafil® for ProTaper Gold®	F1 - F2	Setting 1	20 seconds	± 5 Seconds
F3 - F4 - F5		Setting 2	44 Seconds	± 7 Seconds	

### 4) WARNINGS

Do not touch any hot surfaces. Surfaces that might get hot during operation are marked with warning sign .

When encountering warning sign  consult the instructions for use (this document).  
Disconnect the mains power plug before cleaning the heating chambers.

### 5) PRECAUTIONS

- To prevent fire or electric shock, do not expose this product to moisture.
- The product does not have an ON/OFF switch. In order to disconnect the product from the mains power, unplug the mains power connector.
- Unplug the oven while not in use.
- Do not remove the cover of the oven.
- This oven is designed for use in a clinical or hospital environment. Do not use in any other environment.
- To reduce the risk of electric shock, do not remove cover; there are no user-serviceable parts inside.
- This oven is intended for heating gutta-percha coated obturators only. Do not insert anything other than an obturator into the oven.
- Changes or modifications not expressly approved by the distributor could void the user's authority to operate the equipment.
- Always follow the instructions for use when using the product. Product safety and performance are guaranteed only if the product is used in accordance with the instructions for use and its intended use.

## 6) ADVERSE REACTIONS

In the present technical state, no adverse reaction has been reported so far.

## 7) STEP-BY-STEP INSTRUCTIONS

- 1) Prior to plugging in the mains power plug, inspect oven to assure it is clean. Clean it if necessary (see Chapter 8).
- 2) Turn the power on by plugging the mains power plug into a 115 or 230 VAC outlet, depending on the oven type. Both indicator lights around the buttons will wake up the standby mode, push and hold either one of the buttons until the indicator light around the button is illuminated and two signal “beeps” are heard, and release the button. A second, lower signal “beep” will sound. On both sides the lowest heat setting is selected as shown by indicator lights. The oven is now ready for use. To return to standby mode push and hold either one of the buttons until the indicator light around the button turns off.
- 3) Put both obturator holders in the upper position by pressing the back of the holder (marked area).
- 4) Disinfect the obturators prior to placing them into the holder. Immerse the obturators in 5.25% NaOCl for 1 to 5 minutes at ambient temperature. Remove all the bubbles from the surface of the obturator. Gently wipe the gutta-percha with sterile gauze moistened with 70% isopropyl alcohol. Let the obturators air dry. Do not use disinfecting solutions containing Phenol or any products which are not compatible with the filling material or plastic post.
- 5) After disinfecting the obturator and setting the rubber stop, place the obturator in one of the obturator holders, lightly press on top to have it firmly seated. Regardless of what working length you have determined, the rubber stop must be under the holder. Make sure the obturator is hanging straight, not at an angle. The obturator must clear all sides of the heating chamber.
- 6) Push the button which corresponds to the selected obturator holder, to select the size of the obturator you wish to heat. Repeatedly push this button to select the heat setting. Refer to the table of Chapter 3 for the correct heat setting.
- 7) To start the heating process push the obturator holder down until you hear it click. It will raise a little but keep in a steady down position. The indicator light around the button will show that the heater is on (constantly on). The obturator is now being heated. Heating time varies – depending on obturator size and type – from 20 to 49 seconds, and is regulated automatically.
- 8) After the first signal “beep”, the obturator is ready for use. Push the back of the obturator holder (marked area) to let it raise slowly. The indicator light around the button will go off. Take the obturator carefully out of the holder by pulling it towards you, making sure not to scrape the obturator on any part of the holder.
- 9) You may leave the obturator in the Thermaprep® Obturator Oven for up to 90 seconds after the first signal “beep.” It will keep the obturator at the appropriate temperature and ready for use. The oven will “beep” every 15 seconds to remind you that the obturator is still in the oven. After 90 seconds, the heating element will switch off automatically.
- 10) If you want to heat more obturators, you may use one holder or both to continue your work efficiently. Each holder is operated by its own button and indicators. If you hear a signal “beep” then look at indicator lights around the buttons which indicate obturator heater is ready (flashing). When both heaters have finished their cycle and no buttons are pushed, the oven will go into a power-saving standby mode after 5 minutes.
- 11) Disconnect oven after use and before cleaning.
- 12) Clean oven after each use (see Chapter 8).

## 8) CLEANING INSTRUCTIONS

Do not use decontamination or cleaning agents which could cause a hazard as a result of a reaction with parts of the material contained in it.

Clean oven after each use. Any gutta-percha remaining on the holder or in the heating chamber may be effectively removed with Orange Solvent (generic dental solvent) on a cotton pledget. Press and hold both buttons at once for 2 seconds to put the oven into the cleaning mode. This will cause both heaters to heat up to a medium temperature of 140°F/60°C. The indicator lights around the buttons will show that the heaters are on (constantly on).

A signal “beep” and flashing indicator lights around the buttons will warn that the desired temperature has been reached. Disconnect the mains power plug and clean the heater chambers with a wire brush from bottom side as indicated.

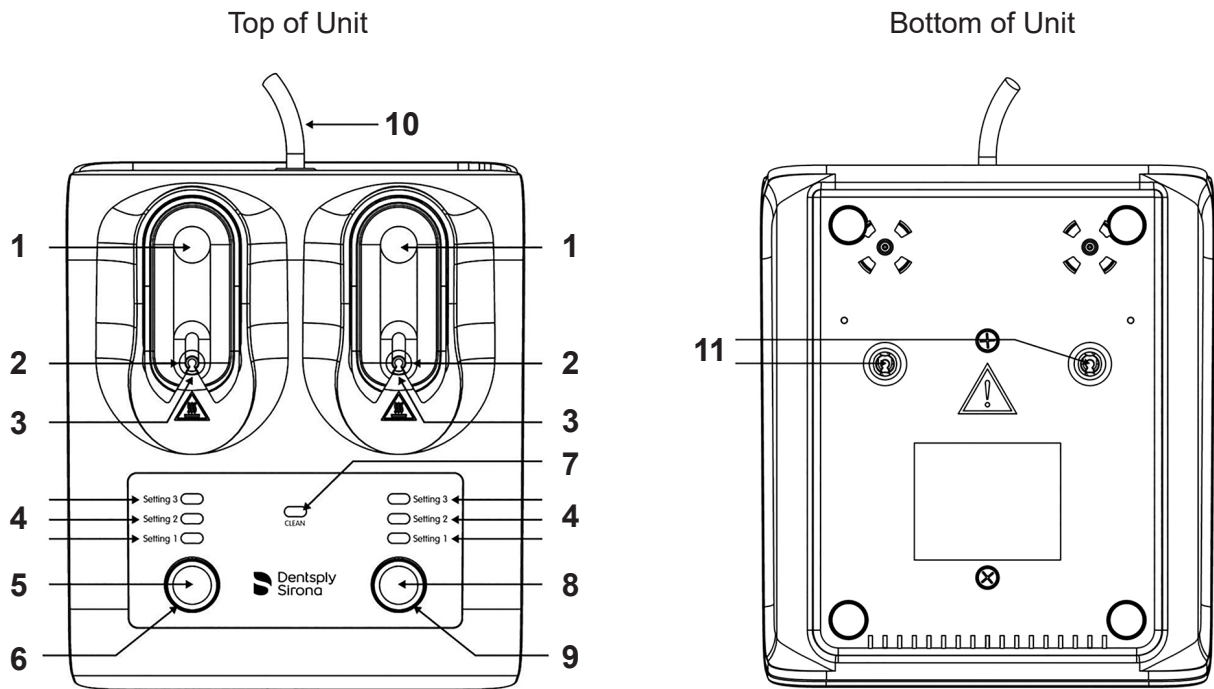
To abort the cleaning mode without disconnecting the mains power plug and to enter the normal mode again press either button for 2 seconds.

If there is any doubt about the compatibility of decontamination or cleaning agents, please contact the cleaning agent’s manufacturer or the distributor.

## 9) TECHNICAL SPECIFICATIONS

Voltage rating	115 or 230 VAC (± 10%), 50/60 Hz
Standby power	<1.5 W
Normal use power	380 W max.
Fuse rating	4AT 250 V (non-replaceable)
Equipment mass	0.56 kg
Equipment dimensions	155 x 178 x 81 mm

The equipment is designed to be safe for indoor use under the following environmental conditions:	
Operating temperature	+5°C to 40°C, +41°F to 104°F
Max. operating humidity	80% at +31°C to 50% at 40°C (lin.), 80% at +87°F to 50% at 104°F (lin.)
Storage temperature	-20°C to +65°C, -4°F to +149°F
Storage humidity	20 to 80%, no condensation
Max. altitude	2000 m
Transient overvoltages	Category II
Pollution degree	2
Mains voltage fluctuations	Up to ± 10% of the nominal voltage
Electromagnetic environment	Basic

- 1) Press here to lower the obturator holder and start heating. Press here again to stop heating and raise the obturator holder.
- 2) Place obturator here.
- 3) Heating chamber.
- 4) Settings 1, 2 and 3.
- 5) Button to select obturator size for left obturator heater.
- 6) LEFT heater "ON" indicator light.
- 7) Cleaning mode indicator light.
- 8) Button to select obturator size for right obturator heater.
- 9) RIGHT heater "ON" indicator light.
- 10) Mains power cord with mains power plug.
- 11) Access hole to heating chambers for cleaning with wire brush.

Technical assistance can be obtained from the distributor, see last page for contact details.

## 10) ADDITIONAL INFORMATION

The oven is designed with two precision heating alerts:

- **First size LED button blinks:** The targeted temperature of the heating element was not reached within specified time (time-out). Use the other heating element to warm the obturator. This may be re-set by disconnecting and then reconnecting the power plug to the mains power. If the error occurs again the oven should no longer be used and sent for repair.
- **Second size LED button blinks:** The hardware safety circuit has been activated due to a failure in the heating element, causing overheating. This may be re-set by disconnecting and then reconnecting the power plug to the mains power. If the error occurs again the oven should no longer be used and sent for repair.

Any serious incident in relation to the product should be reported to the manufacturer and the competent authority of the country in which the serious incident occurred.

Symbols	EN
	Instructions for use translator
	Class II equipment
	Caution: hot surface
	Serial number
	CE marking, compliance with applicable EU legislation
	Alternating current
	Medical device
	Reference number
	Unique device identifier
	Distributor

Symbols	EN
	Do not use if packaging is damaged
	Opened packages are not replaced
	Separate collection for electronic and electrical equipment
	Manufacturer
	Warning! Failure to follow all instructions correctly during use can damage the product or endanger users / patients
	Consult the instructions for use (this document)
	Storage temperature
	Storage humidity
	Fragile, handle with care
	Keep dry
	Country and date of manufacture

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Endotron B.V.